Dreve

Otoplastiken

Gebrauchsanweisung
Directions for use
Mode d'emploi

www.dreve.com

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Intended use

Earmold as a custom-made product for hearing systems.

Indication

Earmold for sound transmission from the hearing aid to the ear.

Patient target group

Patients for whom a fitting of the ear is necessary.

Attention!

These instructions contain additional information for your individual earmold. Please read the operating instructions for your hearing aid carefully.

Product description

The earmold serves as the essential link between the human ear and a hearing aid. It thus enables the best possible hearing aid fitting with the greatest possible wearing comfort and fitting accuracy.

Dreve manufactures each earmold individually and with highest quality standards. The earmold you have purchased therefore complies with the basic requirements listed in Appendix XIII of Medical Device Regulation 2017/745.

RIC: Earmolds for hearing aids with external receiver

Earmolds for micro tube hearing systems

Hearing systems with standard tubing

Use

The hearing aid fitting, the detailed instructions for use, such as the insertion and removal of the earmold, will be carried out by your hearing care professional.

Regular cleaning

The earmold should be cleaned after each use. It is important to check the earmold regularly for contamination, deposits and blockages, as these can affect the sound quality of the hearing aid. For thorough cleaning and disinfection, we recommend our OtoVita® care series. Depending on the hearing system, whether BTE hearing aid with standard tube, micro tube or hearing aid with external receiver, there are different care sets for cleaning. Your hearing care professional will explain to you exactly how to clean your hearing aid most effectively with the earmold.

Precautions

- Do not use any sharp/ pointed objects when cleaning.
- Contact with aggressive cleaning agents and solvents (e.g. acetone) must be avoided!
- Protect the earmolds from extreme temperatures.
- The earmold and the hearing tube must be completely free of moisture before use.

Contraindication

In case of allergies to any of the ingredients do not use this medical device. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

Serious incidents

All serious incidents relating to the device must be notified to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Disposal

Disposal according to local / regional / national / international regulations.

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